



**mdINR**<sup>®</sup>  
App

Quick start user guide



- 1 Download mdINR from the iTunes App Store.

## Log in to the app

- 2 Open the app from your home screen by tapping the icon:



- 3 Log into the app with your Patient ID and PIN and press **Login**.




**TIP:** Your device needs to be connected to the internet to log in. Make sure you have a working cellular or Wi-Fi connection.

# Quick start user guide

## Submit a test result

- ① Complete an INR test with your Roche CoaguChek® XS device in the usual way.
- ② Log into the app as described in this guide.

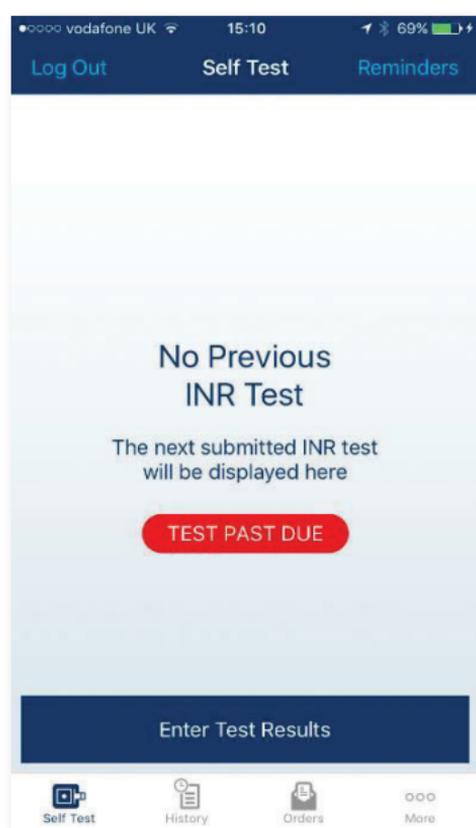

- ③ From the app's main screen, tap **Enter Test Results.**



- 4 Point the phone's camera at the display of CoaguChek® XS device until the app shows a green square around the display of the device.

### TIPS:

You may have to move the phone or CoaguChek XS device to get it into the right position.

As soon as the app detects a valid result, it will progress to the next step automatically.

If you can't get the app to read the results, you can enter them manually by tapping **Enter Manually** at the bottom of the screen.

# Quick start user guide

- 5 The app will show the results on screen once it has detected them. Double-check to make sure they are correct. If everything is OK, press **Submit**.



## TIPS:


If any of the values are wrong, you can edit them before sending them by tapping them.

You will need to be connected to the internet to submit a result. Make sure your phone is connected by cellular or Wi-Fi.



## Set a reminder

The app allows you to set a reminder to notify you when your next test result is due. You can set the reminder to activate on a day and at a time that suits your schedule. When you tap the notification, it will take you straight to the app where you can log in and submit a test result.

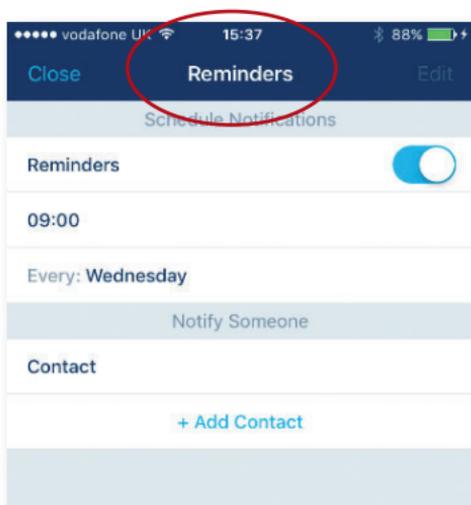

### To set a reminder:

- 1 From the app's main screen, tap **Reminders** in the top right.
- 2 Tap **Select Time** to select a time of day for the reminder.
- 3 Tap **Select Day** to choose a day of the week for the reminder.
- 4 Tap **Reminders** to switch on reminders.

# Quick start user guide

⑤ Tap **Close** in the top left to leave this screen.

⑥ A reminder has now been set.

## **TIP:**

If you want to disable reminders, you can go back into **Reminders** and tap the setting to switch it off again at any time.

## **Respond to a reminder**

① Tap the reminder

② The mdINR app launches. Enter your PIN.

③ Follow the previous instructions to capture the result.

## **Assign a contact to be alerted when you submit your result**

The app allows you to automatically send a notification message to a friend or family member each time you submit a result. This gives them peace of mind that the result has been sent.

## **NOTES:**


To assign a contact, make sure you have them in your device Contacts list.

An active cellular connection is required to send a message. The feature will not work on non-cellular devices such as iPad and iPod Touch.




## Assign a contact to be alerted when you submit your result (continued)

- 1 From the app's main screen, tap **Reminders** in the top right.

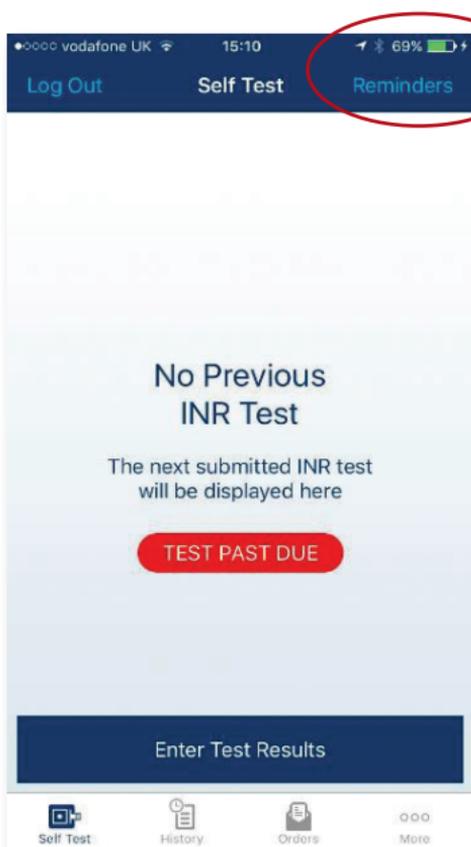


- 2 Under **Notify someone**, select **+ Add Contact**.
- 3 Select a contact from your phone's Contacts book.
- 4 Tap **Close** to return to the app mainscreen.

# Quick start user guide

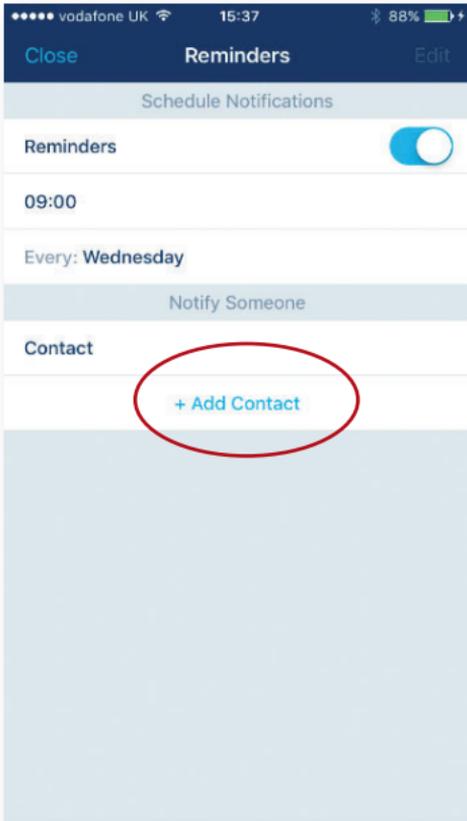

When you submit your INR result, the person selected will get a message “[Your name] has submitted their INR result”

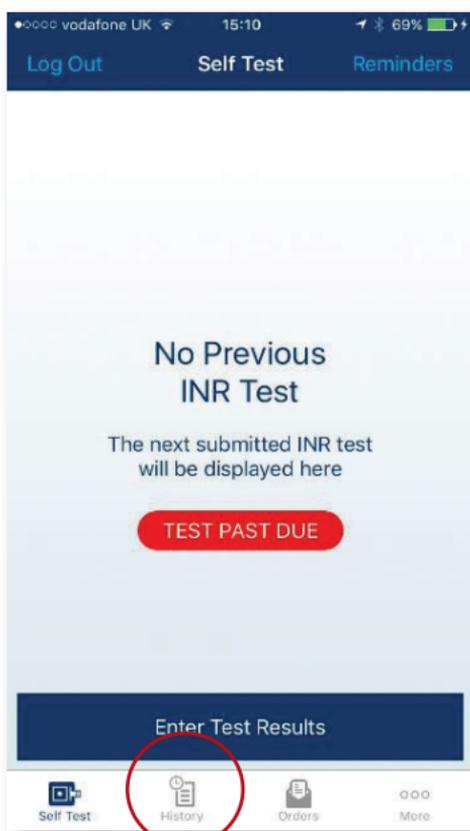

## View previous results

You can view previous results submitted to mdINR.

- 1 From the app homescreen, tap **History** at the bottom of the page.
- 2 The app will show a list of the previous results.

# Quick start user guide



- ③ To view the results as a graph, turn your device so it's in a landscape orientation.




- ④ To return to the list view, turn your phone back to a portrait orientation.
- ⑤ You can return to the app's main screen by tapping **Self Test** at the bottom.

# Quick start user guide

## Help & troubleshooting

If you need help with the app or device, you can read answers to frequently asked questions and contact mdINR via the app.

- 1 Tap **More** in the bottom right of the app homescreen.

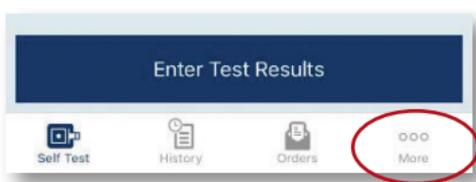

- 2 To read answers to frequently asked questions, tap **FAQs**.

- 3 To contact mdINR tap **Call mdINR** or **Email mdINR**.

### NOTES:

A cellular connection is required to make a phone call. Your operator may charge you for the call.

A data connection (cellular or Wi-Fi) is required to email mdINR.

